

Pepsin Solution

INTENDED USE

For In Vitro Diagnostic Use

<u>AVAILABILITY:</u>	<u>Catalog #</u>	<u>Volume</u>
	TA-125-PE	125 ml (1250 slides)

<u>SPECIFICITY:</u>	N/A
<u>ENZYME:</u>	N/A
<u>CHROMOGEN/SUBSTRATE:</u>	N/A

DESCRIPTION

Pepsin is a commonly used digestive enzyme for immunohistochemical procedures. This product is provided as a single component, ready-to-use solution. The pH of the pepsin solution is 2.0.

WARNINGS & PRECAUTIONS

Refer to MSDS.

STORAGE & SHELF LIFE

Store at 2-8°C. Each component is stable for 18 months.

MICROBIOLOGICAL STATE

Product(s) not sterile.

MATERIALS REQUIRED BUT NOT PROVIDED

N/A

SPECIMEN & REAGENT PREPARATION

Refer to Procedure.

PROCEDURE

1. Incubate tissue section for 10 minutes at 37°C.
2. Rinse and proceed with immunostaining.

REFERENCES

N/A

TROUBLESHOOTING

Please contact Thermo Fisher Scientific Technical Support by phone (1-510-991-2800 or 1-800-828-1628) or by email (lab.reagents@thermofisher.com).