

Biotinylated Goat Anti-Polyvalent (Ready-To-Use)

INTENDED USE

For In Vitro Diagnostic Use

<u>AVAILABILITY:</u>	<u>Catalog #</u>	<u>Volume</u>
	TP-060-BN	60 ml
	TP-125-BN	125 ml

<u>SPECIFICITY:</u>	Anti-Mouse IgG (H+L), Anti-Rabbit IgG (H+L)
<u>ENZYME:</u>	Biotin
<u>CHROMOGEN/SUBSTRATE:</u>	N/A

DESCRIPTION

This reagent is intended to be used in a labeled streptavidin-biotin immunoenzymatic antigen detection system. This technique involves the sequential incubation of the specimen with an unconjugated primary antibody specific to the target antigen, a biotinylated secondary antibody that reacts with the primary antibody, enzyme-labeled streptavidin, and substrate-chromogen.

WARNINGS & PRECAUTIONS

Refer to MSDS.

STORAGE & SHELF LIFE

Store at 2-8°C. Each component is stable for 18 months.

MICROBIOLOGICAL STATE

Product(s) not sterile.

MATERIALS REQUIRED BUT NOT PROVIDED

N/A

SPECIMEN & REAGENT PREPARATION

Refer to Procedure.

PROCEDURE

STAINING PROTOCOL (kit components in bold):

1. Deparaffinize and rehydrate tissue section.
2. (Optional) To reduce nonspecific background staining due to endogenous peroxidase (only when using peroxidase label), incubate slide in hydrogen peroxide for 10-15 minutes.
3. Wash 2 times in buffer.
4. If required, incubate tissue in digestive enzyme (or appropriate pretreatment).
5. Wash 4 times in buffer.
6. (Optional) Place slide in Ultra V Block and incubate for 5-10 minutes at room temperature to block nonspecific background staining. **NOTE:** Do not exceed 10 minutes or there may be a reduction in desired stain.
7. Rinse (Optional).
8. Apply primary antibody and incubate according to manufacturer's protocol.
9. Wash 4 times in buffer.
10. Apply **Biotinylated Goat Anti-Polyvalent** and incubate for 10-15 minutes at room temperature.
11. Wash 4 times in buffer.

12. Apply enzyme labeled streptavidin and incubate according to manufacturer's protocol.
13. Rinse 4 times in buffer.
14. Apply appropriate chromogenic substrate and incubate until desired reaction is achieved.
15. Counterstain and coverslip.

NOTE: This reagent is provided in bulk form for use with automated systems (including capillary gap methodologies) or in staining jars for repeated use. If the DIP method is employed, staining jar should be sealed and refrigerated between uses. Remove staining jars from the refrigerator 30 minutes prior to staining to allow reagents to come to room temperature.

The specificity and sensitivity of antigen detection is dependent on the specific primary antibody used.

REFERENCES

N/A

TROUBLESHOOTING

Please contact Thermo Fisher Scientific Technical Support by phone (1-510-991-2800 or 1-800-828-1628) or by email (lab.reagents@thermofisher.com).